
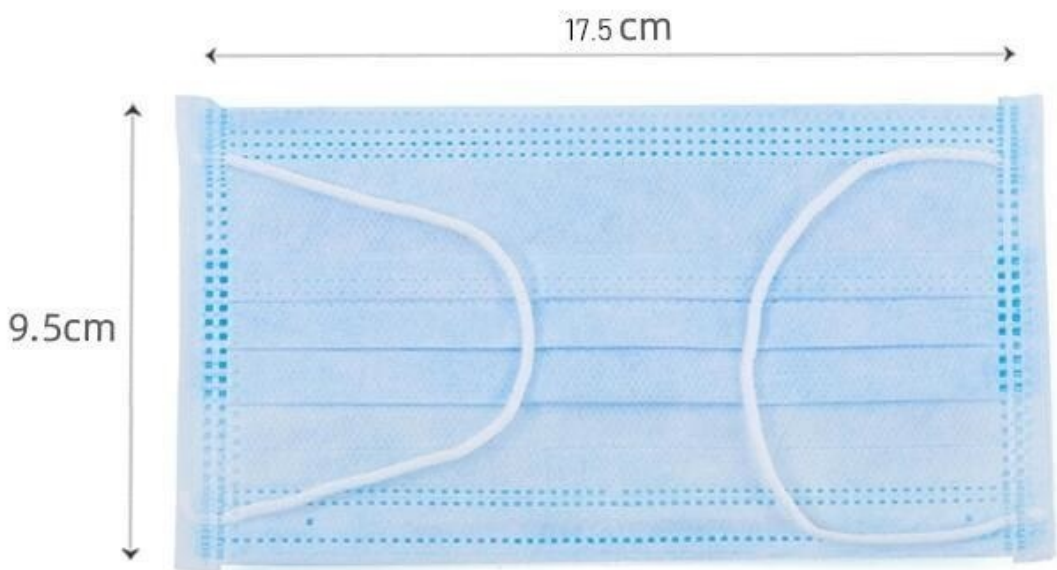
## Product description

### Non Woven Disposable Face Mask 3 Layers Medical Earloop Masks

1. Filtering rate:  $\geq 95\%$
2. Size: 17.5\*9.5cm (for adult)
3. Material: Non-woven fabric + Melt blown fabric
4. Color: Blue/White
5. Package: 50pcs per pack
6. Type: Earloop + Adjustable nose clip
7. Storage condition: Relative humidity  $< 80\%$
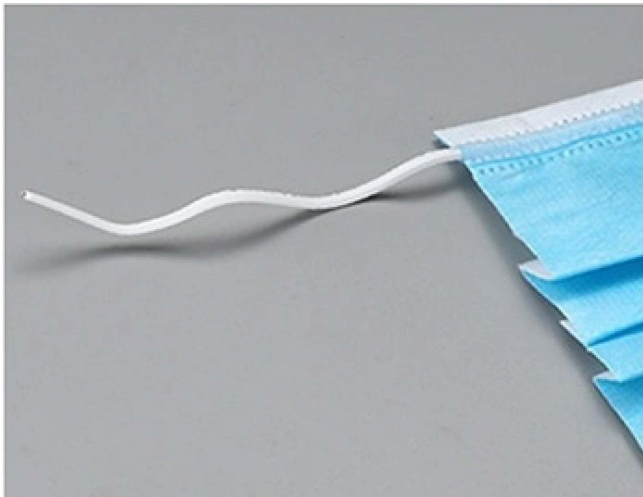
## Product details


## Magic nose stickers

Adjust the fit with the face to ensure the protection effect


## High elasticity and tightness

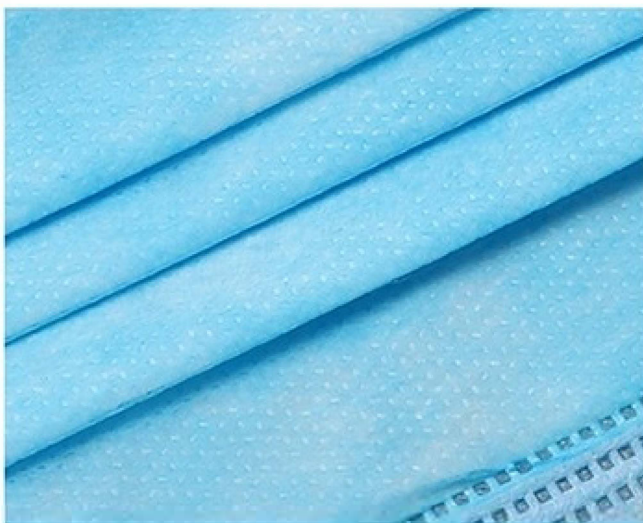Loose elastic band for a more comfortable fit

## Folding design

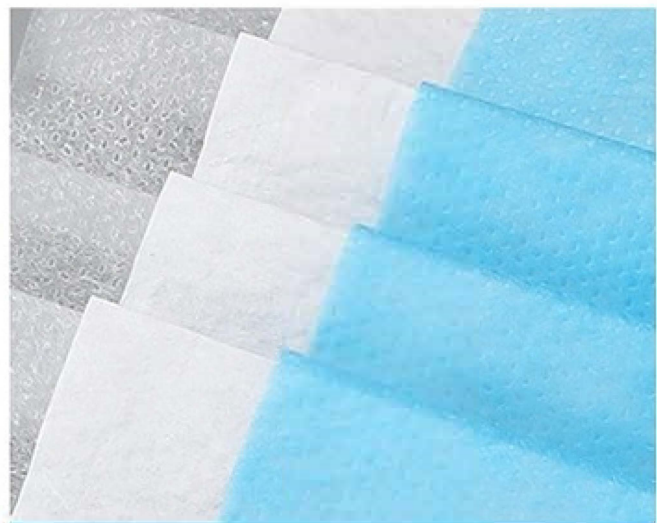Three layer folding design, large space for more ventilation

## Three layer filter design

Use non-woven fabric, melt blown cloth and other filter materials

# Three structure effective protections

**Skin-friendly nonwovens**

Good touch soft skin

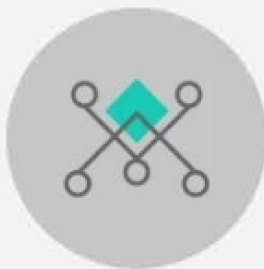
**Outer nonwovens**

Filtering larger particles of drop dust

**Blocking the melting cloth**

Meltcloth fiber tissue fine, the main material is high melting fiber polypropylene, after electrostatic treatment to achieve filtration efficiency


**High-density isolation and purification of intake air**

**formaldehyde**

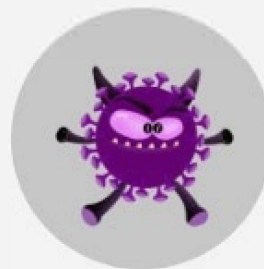**dust**

**virus**

# Dustproof waterproofing

The high quality adhesive-bonded  
cloth material quality breath resistance  
slightly has the small isolation

## User guide

**1**  
Put your fingers through the elastics.  
Nose bar should be found above. Place the mask on your nose and mouth.

**2**  
Put the elastics on your ears.  
Pull the mask by its upper and lower edges to fully unfold it. That will ensure maximum face protection and minimize the number of layers you need to breathe through.

**3**  
Put on and form a stiffener above the bridge of the nose to minimize air leakage.

**4**  
Take off the mask by grasping the elastics and pulling them off your ears. Do not touch the mask while removing - it may contain germs.

Dispose the mask after use. Wash your hands carefully.

## Packing design

# EC Declaration of Conformity

We, Linyi Yuteng Impor&Export Co., Ltd. Hereby declare that below product:

Medical Masks

UMDNS code 12447

GMDN code 35177

Lot IT20201104

Reference standard EN 14683: 2019

Meets the provisions of Directive 93/42/EEC which apply it.

According to Annex IX of the Directive 93/42/EEC, Rule 1, this medical device has been assigned to

## Class I

It bears the mark

This Product has been manufactured under a quality management system according to Annex VII of Directive 93/42/EEC.

This Declaration of conformity is valid in connection with the release document of the respective batch of Medical masks.

The above mentioned declaration of conformity is exclusively under the responsibility of Linyi Yuteng Impor&Export Co., Ltd.

Manufacturer Name: Linyi Yuteng Impor&Export Co., Ltd.

Address: No.1 Yihe Five Road, Linyi Comprehensive free Zone, Linyi City, Shandong Province, China

Zip code: 276000 Tel: +86 539 7701371

Authorized Representative:

Name: Mibsa S.r.l.

Address: Via Melo da Bari 102, 70121 Bari, Italy

info@mibsa.com

Linyi, 04/16/2020

Place Date of Issue

A handwritten signature in black ink, appearing to read 'Crispianjun', written over a horizontal line.

Legally binding signature, Functionary

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